<p>Do not use if pouch has been damaged or opened. Niet gebruiken indien de verpakking beschadigd of geopend is. Nicht verwenden, wenn der Beutel beschädigt oder geöffnet ist. Não utilizar se a embalagem tiver sido danifi cada ou aberta. Non utilizzare se la busta è stata aperta o danneggiata. אין להשתמש אם השקית נפגמה או נפתחה. ห้ามใช้หากถุงบรรจุหรือถูกเปิดทำลาย 如果袋子已损坏或打开，请勿使用。 لا تستخدم الضماد إذا كان الكيس تالفاً أو مفتوحاً. 주머니가 손상되었거나 열려 있을 경우에는 사용하지 마십시오. Nepoužívajte, ak je vrecúško poškodené alebo otvorené. Ne pas utiliser si l’emballage du pansement a été ouvert ou est endommagé. No utilizar si el estuche está dañado o abierto. Ne uporabljajte, če je ovojnina poškodovana ali odprta. Μη χρησιμοποιείτε εάν το σακουλάκι έχει υποστεί ζημιή ή έχει ανοίξει. Nie używać, jeżeli saszetka została otwarta lub uszkodzona. Eğer poş zarar görmüş veya açılmış ise kullanmayın.</p>		<p>Keep away from sunlight and heat. Uit de buurt van zonlicht en warmte houden. Vor Sonnenlicht und Hitze schützen. Manter longe da luz solar e calor. Tenere lontano dalla luce solare e dal calore. יש להרחיק מאור השמש ומחום. เก็บให้ห่างจากแสงแดดและความร้อน 远离阳光和热。 تُحفظ بعيداً عن لئمة الشمس والحرارة. 햇빛과 열에 노출시키지 마십시오. Udržavajte mimo dosahu priameho slnečného svetla a tepla. Garder à l'abri des rayons du soleil et de la chaleur. Mantener fuera del alcance de la luz solar y el calor. Hranite zaštiteno pred sončno svetlobo in vročino. Przechowywać z dala od źródła ciepła i światła słonecznego. Güneş ışığı ve ısıdan koruyun.</p>	
<p>Store ≤30°C (86°F). Bewaren ≤30°C (86°F). Bei ≤30°C (86°F) lagern. Armazenar em ≤30°C (86°F). Conservare a ≤30°C (86°F). שי אחסון בטמפרטורה ≤ 30°C (86°F) เก็บในที่มีอุณหภูมิไม่ควรเกิน 30°C (86°F) 儲存溫度 ≤ 30°C (86°F) التخزين ≤ 30 درجة مئوية (86 فهرنهايت). 천연고무 라텍스로 만든 제품이 아닙니다. Vyrobené bez prírodného latexu. Ne contient pas de latex naturel. No están hechos de látex de caucho natural. Ne vsebuje naravnega lateksa. Δεν κατασκευάζεται από φυσικό καουτσούκ. Bez lateksu z kauczuku naturalnego. Doğal kauçuk lateksten üretilmemiştir.</p>	<p>Not made with natural rubber latex. Niet gemaakt met natuurlijke rubberlatex. Nicht aus Naturkautschuklatex hergestellt. Não é feito com látex de borracha natural. Privo di lattice di gomma naturale. מיצור ללא גומי לטקס טבעי. ไม่ใช้ผลิตภัณฑ์ยางธรรมชาติ 不是采用天然橡胶胶乳制成。 غير مصنوع من لاتكس المطاط الطبيعي. 천연고무 라텍스로 만든 제품이 아닙니다. Vyrobené bez prírodného latexu. Ne contient pas de latex naturel. No están hechos de látex de caucho natural. Ne vsebuje naravnega lateksa. Δεν κατασκευάζεται από φυσικό καουτσούκ. Bez lateksu z kauczuku naturalnego. Doğal kauçuk lateksten üretilmemiştir.</p>	<p>Single use. Do not re-use. Eenmalig gebruik. Niet hergebruiken. Zum Einmalgebrauch. Nicht wiederverwenden. Uso único. Não reutilizar. Monouso. Non riutilizzare. לישמוש חד-פעמי. אין לעשות שימוש חוזר. ใช้เพียงครั้งเดียว ห้ามนำกลับมาใช้ใหม่ 一次性使用。请勿重复使用。 لاستخدام لمرة واحدة. لا تُعد استخدام الضماد. 일회용. 재사용하지 마십시오. Len na jedno použitie. Nepoužívajte opakovane. À usage unique. Ne pas réutiliser. Desechables. No reutilizar. Samo za enkratno uporabo. Ne uporabljajte ponovno. Μίας χρήσης. Μην επαναχρησιμοποιείτε. Bez jednorazowego użytku. Nie używać ponownie. Tek kullanılmıktır. Tekrar kullanmayın.</p>	 

NL

## PolyMem® Kompres of Windsel

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## EN

## PolyMem® Pad or Roll Dressing

The proven multifunctional dressing.

**DESCRIPTION:**
PolyMem polymeric membrane dressings are made of moisture-loving polyurethane matrix that contains (1) safe wound cleanser, (2) glycerin moisturizer, and (3) superabsorbents. Dressings have a thin polyurethane backing which breathes and helps keep out contamination. Dressings may be used as a primary, secondary, or combined primary and secondary dressing. PolyMem MAX™ dressings have the same polymeric membrane as PolyMem dressings, but MAX is thicker and has higher MVTR to handle more fluid.

Drug-free PolyMem dressings help to:

- effectively manage and heal wounds
- absorb fluid and provide a moist healing environment
- minimize and relieve persistent and procedural wound pain
- reduce edema, bruising, and the spread of inflammation into surrounding undamaged tissues

**INDICATIONS FOR ALL HUMAN BEINGS:**

**Over-the-counter**, for management of minor wounds, such as:

- Abrasions • Lacerations • Cuts • Burns

**Under the direction of a healthcare professional**, for the management of serious wounds, such as most partial and full thickness wounds, including:

- Pressure ulcers (stages I-IV)
- Venous stasis ulcers
- Acute wounds
- Leg ulcers
- Donor and graft sites
- Skin tears
- Diabetic ulcers
- Dermatologic disorders
- 1st and 2nd degree burns
- Surgical wounds
- Vascular ulcers

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## EN

**PRECAUTIONS:**

- For external use only
- PolyMem dressings are not compatible with oxidizing agents (hydrogen peroxide or hypochlorite solutions)
- Prior to performing imaging procedures, determine if the dressing should be removed
- Avoid contact with electrodes or conductive gels
- Topical treatments are not recommended in conjunction with PolyMem dressings
- Be alert for signs of infection or maceration
- Do not use and discontinue use on people who show signs of sensitivity, irritation, or allergy from the dressing or its materials
- Intended for one use on one person. Do not reuse. Reuse may degrade properties, spread contamination, and increase risk of infection.

**PolyMem dressings may be used when signs of infection are present if proper medical treatment that addresses the cause of the infection has been implemented.**

**Store** at room temperature ≤30°C (86°F). Excursions permitted ≤55°C (131°F) but should be minimized.

**Dispose** as typical for biohazardous waste. **Dimensions** are approximate.

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## EN

**DIRECTIONS:**
**Initial Application:**

- Prepare the wound according to protocol or as directed by a physician or other ordering clinician.
- Select a dressing with the membrane pad ¼” – 2” (0.6 – 5 cm) larger than the wound. The dressing should also cover any inflamed or damaged area surrounding the wound. Dressings may be cut.
- For dry wounds, moisten dressing slightly prior to application.
- Apply film side out (so printing is visible).
- Secure dressings, such as with tape, netting, or gauze.
- Outline the wound on top of dressing to help determine when to change.

**Use:**

- A dramatic increase in wound fluid may be observed during the first few days due to the fluid attraction. This is not uncommon and indicates that the dressing is working.
- Keep the dressing dry and in place when bathing. Change the dressing if it gets wet.

**Dressing Change:**

- For an exuding wound, fluid will become visible through the top. Change before fluid reaches the wound margin, or when good practice dictates, or after no more than 7 days. Change immediately if fluid reaches the edge of the membrane pad.
- More frequent changes may be needed due to a compromised immune system, diabetes, infection, hypergranulation, macerated tissue, or when desiring to facilitate the removal of non-viable tissue.
- Gently remove the dressing. The PolyMem pad will generally not adhere to the wound bed, usually ensuring pain-free dressing changes. Inspect the dressing and wound to assure removal of all material.

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## EN

## EN

- If coagulated blood or dried exudate has caused the dressing to adhere to skin or hair, use a small amount of saline or water to slowly gently soften and loosen. Clean the intact skin around the wound.
- Manage dressing changes according to protocol or as directed by a physician or other ordering clinician. In most cases, when using PolyMem, there is no need to disturb or cleanse a wound during changes unless the wound is infected or contaminated. PolyMem is designed to continuously clean the wound and does not leave residue that needs to be removed.

**Excessive cleaning may injure regenerating tissue and delay wound healing.**

- Apply a new dressing.

**Observations during use or change:**

- During normal healing, the wound may appear to become larger during the first few dressing changes because the body is naturally removing non-viable tissue. If the wound continues to become larger, consult an expert.
- PolyMem dressings help support new blood vessel formation so it is not uncommon to see blood-stained wound fluid and dressings during changes.
- Be alert for signs and symptoms, such as, increased redness, inflammation, pain, warmth, odor, whitening, fragility, excessive tissue formation, or lack of healing. Consult a professional for problem situations, such as, infection, irritation, sensitivity, allergy, maceration, or hypergranulation.

Any serious incident that has occurred in relation to this device should be reported, by the user and/or patient, to Ferris Mfg. Corp. and the competent authority of the Member State in which the user and/or patient is established.