B▶ WISE

EN

INSTRUCTION FORUSE

BWISESuper Absorbent DressingAdherent

Description

BWISE &per Absorbent Dressing Mherent is an absorbent wound dressingthat helps to maintain a moist environment at thewound surface. It comprisesan absorbent coridor absorb and retain exudates, the absorbent core is enclosed in hydrophobic breathable layer and liquid permeablelayer, and the sidefacing thewound it is covered with a silicone wound contact layer, Super Absorbent Dressing absorbs and retains exudates and MMPs into the absorbent core. to reduce the risk for maceration, and encouragewound healing. The gentle silicone allows for easy application, removal and reapplication on the same area, and it can minimize pain to patients and trauma to wound and its surroundingskin.

Celling material contents: Polypropylene, Polyethylene, Cellulose, Polycrylate, Silicone.

Indications

BWISE Super Absorbent dressing is intended for used in dermalate, to heavily exudate wounds, such as: leg ulcers, pressure ulcers, diabetic foot ulcers and surgical wounds.

Instructions for use

Super absorbent dressing can be used by health care professionals and lay persons under supervision of health care professionals.

1. After wound is cleaned and sterilized, the appropriate size dressing is selected to allow for sufficient space to thewound edges.

2. BWISE Super absorbent dressing can be used both as a primary dressingsor asa secondarydressing. When used as a primarydressing, BWISESuper absorbent dressing must bein close contact with the wound bed to ensure wound fluid is absorbed. Secure accordingly with bandage, tapes or compression.

3. Thefrequency of changingdressingdepends on the amount of exudates.

4. Dressing removal can be facilitated by moistening the dressing with saline solution.

Precautions andwarning

1. The product is single use Reprocessingdevices, in orderto reuse them, may seriously damage their integrity and their performance, and Cross-contamination mayoccur.

2. Sterile. Do not use if sterile packaging is damaged or opened prior to use. Do not re-sterilize.

3. Do not cut the dressing, the cover must remain intact.

4. In the absence of available data supporting the use ofthis dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups this dressin should be used with caution and following a clinician's recommendation

Contraindications

1. Do not usethis product on dryor low exudIngwounds, eyes, mucous membranes or in wound cavities because the dressing swells duringabsorption

2. Do not use on third- orfour th burns.

3. BWISE Super absorbent dressing should not be used on persons with known allergy ar hypersensitivity to any of its components.

Storage

Keep in a clean and dry place in specified temperature range and awayfrom sunlight.

Disposal

Dispose should be handled accordngto local environmental procedures.

Other information

If any serious incident has occurred in relation to the use of BWISE Silicone Superabsorbent Dressing, it should be reported to the manufacture and competent authority of the Member States

Shelf life: 3 years from date of manufacture

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